

Instruction for use

Chondrogel

Product Description

Chondrogel is a transparent and sterile gel supplied in a glass applicator. The product is for single use only.

Chondrogel is a unique form of 20 mg of collagen tripeptide dissolved in (CTP) 20 mg of Hyaluronic Acid for viscosupplementation of joints.

Each carton contains 1 prefilled applicator of 2ml Chondrogel. The volume of each applicator is shown on the packaging carton as well as on each applicator.

Product Composition

20 mg of collagen tripeptide dissolved in 20 mg hyaluronic acid.

Indications and Usage

Chondrogel is indicated for degenerative joint diseases (DJD) and the support of joint mobility through viscosupplementation.

Dosage and Method of Administration

Chondrogel administration should be performed exclusively by veterinarians.

Depending upon joint size, up to 4 ml may be administered.

The recommended application protocol is 1 application per 2 to 6 months.

Follow up is recommended at 2-3 weeks.

Use a strict aseptic technique. Discard any unused portion of the applicator.

To use the prefilled applicator, remove the Luer lock cap, attach it to a suitable cannula (recommended is 21 – 25 G) and secure it by turning slightly.

Precautions

Avoid using Chondrogel with instruments sterilized with quaternary ammonium salts solutions.

Warnings

The applicator is intended for single use.

Do not use Chondrogel after the expiry date printed on the package.

The expiry date refers to the products stored properly in its original package.

Do not use Chondrogel if there are visible signs of an open or damaged package.

Keep out of reach of children.

Instruction for use

Chondro gel

Storage

Store at room temperature. Protect from sunlight.

Shelf life is 24 months under recommended conditions.

